

**CORTEZ DIAGNOSTICS, INC.**

23961 Craftsman Road, Suite E/F,  
 Calabasas, CA 91302 USA  
 Tel: (818) 591-3030 Fax: (818) 591-8383  
 E-mail: onestep@rapidtest.com  
 Web site: www.rapidtest.com

**OneStep HBsAb RapiDip InstaTest**

Cat. No.176582-1  
 Cat. No.176583-1

**INTRODUCTION**

One Step Strip Style ANTI-HBs Test is a rapid direct binding test for the visual detection of antibodies to hepatitis B surface antigen (anti-HBs) in serum as an aid in the diagnosis of hepatitis B infection. One Step ANTI-HBs Test is based on the principle of sandwich immunoassay for determination of anti-HBs in serum. Purified recombinant antigens are employed to identify anti-HBs specifically. This one step test is very sensitive and only takes about 10-20 minutes. Test results are read visually without any instrument.

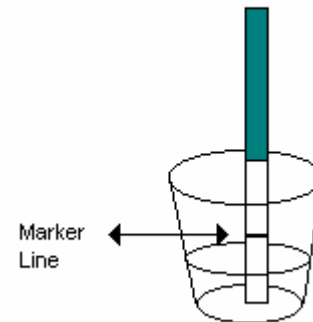
**SPECIMEN COLLECTION**

For serum, collect blood into a container without anticoagulant. Allow the blood to clot and separate the serum from the clot. Use the serum for testing.

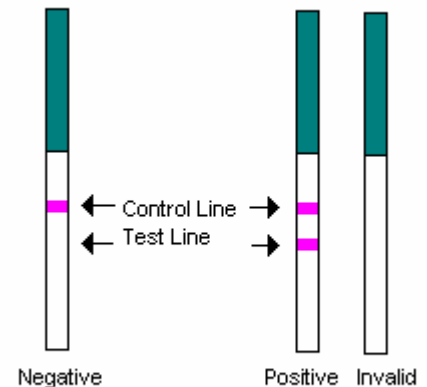
If the specimen cannot be tested on the day of collection, store the serum specimen in a refrigerator or freezer. Bring the specimens to room temperature before testing. Do not freeze and thaw the specimen repeatedly.

**TEST PROCEDURE**

Following the illustration, dip the test strip with arrowside pointing down into the vessel of serum for about 10 seconds. Do not immerse past the marker line. Wait for 10 minutes and read results. Do not read results after 30 minutes.

**INTERPRETATION OF RESULTS**

- **Negative:** Only one colored band appears on the control region. No apparent band on the test region.
- **Positive:** In addition to a pink colored control band, a distinct pink colored band will also appear in the test region.
- **Invalid:** A total absence of color in both regions is an indication of procedure error and/or the test reagent has been deteriorated.

**STORAGE AND STABILITY**

The test kit can be stored at room temperature (18 to 30°C) in the sealed pouch to the date of expiration. The test kit should be kept away from direct sunlight, moisture and heat.

**PRECAUTION**

1. For in vitro diagnostic use only.
2. Do not use test kit beyond expiry date.
3. The test device should not be reused.



**CORTEZ DIAGNOSTICS, INC.**

23961 Craftsman Road, Suite E/F,  
Calabasas, CA 91302 USA  
Tel: (818) 591-3030 Fax: (818) 591-8383

**ISO 13485-2003**